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CE I Class Medical Device
according to Directive 93/42/EEC and further modifications

1. CODE

CPA205 Pro-wormer cover

2. INTRODUCTION

Thank you for purchasing a **ARDEA ONE product by Moretti** product range. It's design and its quality guarantee to the user comfort, safety and reliability. The product have been projected in Italy from the research and development department of Moretti S.p.A. Italia, in order to meet your requirements in inner and outer spaces. The material used, the accessories and the quality test are continuously checked by Moretti staff. Only the production is made in PRC according to the highest quality an safety standards in accordance with the European Directive on Medical devices. This user's manual provides some suggestions as how to correctly use the product you have chosen and gives some valuable advice for your safety. Please read through the manual carefully before using the product. Should you have any queries, please contact your retailer for advice and assistance. In case of doubts, we recommend to contact the dealer who can help and advice you properly.

NOTE check all parts for shipping damage and test before using. In case of damage, do not use the device. Contact your dealer for further instruction.

3. INTENDED USE

The Pro-wormer cover is an accessory for medical devices like wheelchairs. You can use it on all kinds of wheelchair.

WARNING!

- Do not use the product for a purpose not indicated in this manual
- Moretti S.p.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product
- The manufacturer reserves the right to change the information contained in this document without notice

4. DECLARATION OF CONFORMITY CE

Moretti S.p.A. hereby declares, under its own exclusive responsibility, that the products manufactured and sold by Moretti S.p.A. and belonging to the wheelchair category are compliant with the applicable provisions outlined in the Medical Devices Directive 93/42/EEC and further modifications.

To this end Moretti S.p.A. guarantees and declares as follows, under its own responsibility:

1. The devices in question satisfy the essential requirements of Annex I of directive 93/42/EEC as prescribed by Annex VII of the said directive
2. A complete list of the concerned devices in mentioned in the enclosure A of the declaration
3. The devices in question **ARE NOT MEASUREMENT DEVICES**
4. The devices in question **ARE NOT INTENDED FOR CLINICAL INVESTIGATION**
5. The devices in question are sold in **NON STERILE** packaging
6. The devices in question should be considered as belonging to class I
7. **DO NOT** use/install the devices in question for any other purpose than that expressly indicated by Moretti S.p.A.
8. Moretti S.p.A. maintains and submits the documentation certifying conformity with Directive 93/42/CEE and further modifications to the proper authorities

5. GENERAL WARNING

- DO NOT use this product or any available optional equipment without first completely reading and understanding this instruction manual. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before using this equipment otherwise injury or damage may occur
- Do not smoke and / or contact with heat sources, danger of flammability of the product
- Keep the packaged product away from any heat source
- SERVICE LIFE- the aid use limit is defined by the wear of the parts
- ALWAYS keep hands and fingers clear of moving parts to avoid injury
- DO NOT allow children to play on or operate the wheelchair

6. GENERAL DESCRIPTION

6.1 Parts view

CPA205 Pro-wormer cover

- Cover - 1 pc
- Bag - 1 pc

7. ASSEMBLING / DISASSEMBLING


The product can be opened easily, without tools, just following a few manual operations.

NOTE make reference to safety cautions in this manual.

7.1 Assembling

CPA205 Pro-wormer cover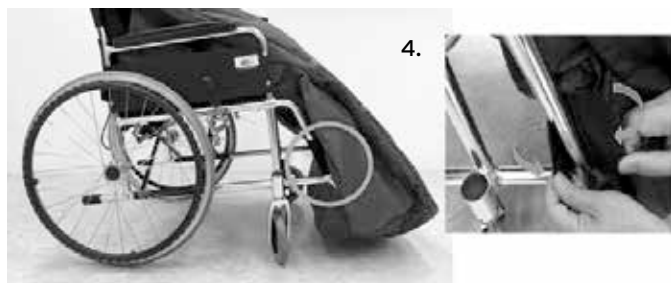

1. Take out the Pro-Warmer from the bag
2. Put the loops through two-side handrails of the wheelchair
3. Open the bottom Velcro and wrap-around footrest pedal and get the Velcro closed
4. Fasten the both sides of lateral Velcro on bottom tubing of the wheelchair
- 5-6. Pro-Warmer assembling completed, zipper open/closure for easy in and out

8. BEFORE USE

- Always check the wear of the parts where the accessory will be fixed to ensure the use of the device in total safety for persons and property.
- Check the correct assembly of the product. Ensure that all parts are properly assembled.
- After completing the assembly, verify the correct operation of the product.
- The rain cover is equipped with side openings, thanks to which you can also use the wheelchair self-propelling, be careful of entrapment due to the reduction of visibility.
- Adapt the product to the user and to the wheelchair you are using.

9. CLEANING AND DISINFECTION

9.1 Cleaning

The cover is in Nylon and upholstery. Clean with warm water and mild soap.

WARNING!

- We recommend a gentle wash at 30 °C
- Do not use bleach
- Do not dry clean
- Do not dry on direct sunlight
- Do not roll or crease
- Do not iron

9.2 Disinfection

If you need to disinfect the product using a common disinfectant cleaner.

NOTE The use of acidic, alkaline or solvents such as acetone or diluents is forbidden.

10. GENERAL CONDITIONS OF DISPOSAL

In case of disposal don't use the inserting container for municipal waste. We recommend to dispose the wheelchair in the appropriate disposal areas for recycle.

11. TECHNICAL FEATURES

11.1 Dimensions and weights

CPA205 Pro-wormer cover

- Height **50cm**
- Width **117cm**
- Depth **40cm**
- Weight **1,5Kg**

11.2 Technical specifications

CPA205 Pro-wormer cover

- Adaptable to any wheelchair
- Outer shell 100% Nylon
- Internal lining 100% polyester fleece
- Recommended storage: Store in a cool and dry place and always away from direct heat sources such as sunlight. (Temp. Recommended between +10 ° C / + 40 ° C)

12. WARRANTY

Moretti products are guaranteed from material or manufacturing faults for 2 years from the purchasing date, except possible exclusion or restriction as follows. The warranty shall not be applied in the possible damages caused by improper use, abuse or alteration, and for the warranty will not be valid if the instructions for use are not strictly adhered to. The correct intended use is specified in this manual. Moretti is not responsible for consequent damages, personal injuries or whatever caused by or in relation to mistaken installation or improper use. Moretti warranty does not cover damages resulting from: natural disaster, not authorized maintenance or repairs, faults caused by problems on electricity supply (when necessary), use of spare parts not covered by Moretti, improper use, alteration not authorized, shipment damages (different from original Moretti shipment), or in case of insufficient maintenance as indicated in the manual. The warranty doesn't cover components subject to wear and tear during the correct use of the device.

12.1 Repairing

Warranty repair: If a Moretti item presents material or manufacturing faults during the warranty period, Moretti will confirm with customer if the fault can be cover from warranty. Moretti, at its unquestionable discretion, can repair or replace the item, by a Moretti dealer or to Moretti headquarters. Labor cost can be charged to Moretti if the repair is covered by warranty. A repair or a replacement doesn't extend the warranty.

Repair not cover by warranty: A product out of warranty can be sent after Moretti authorization. The labor and shipping costs for good out of warranty are to be paid by the customer or by the dealer. The repairs are guaranteed for 6 months from the good received. **Non defective device:** The customer will be informed if, after the device return and

examination, Moretti declares that the device is not faulty. In this case the good will be sent back to customer, the shipping charge will be to be paid by the customer.

12.2 Spare parts

The original Moretti spare parts are guaranteed for 6 months from the receiving date.

13. EXEMPT CLAUSES

Except what clearly specified in this warranty and in accordance with the law, Moretti S.p.A. does not offer its clients any other declaration, warranty or condition, expressed or implicit, including any declaration, warranty or condition of merchantability, of non violation or non interference, of suitability for a specific aim. Moretti S.p.A. does not guarantee that the use of Moretti products will be continuous and faultless. The duration of possible implicit warranties that may be applied in accordance with the law is limited to the warranty period. Some states or countries do not allow limitations to the duration of an implicit warranty, or the exoneration, or the limitations of accidental or indirect damages related to products for consumers. In such states or countries some exonerations or limitation of this warranty may not be applied to the user. This warranty is subject to modifications without any prior notice.

WARRANTY CERTIFICATE

Product _____

Purchased on (date) _____

Retailer _____

Address _____ **Town/city** _____

Sold to _____

Address _____ **Town/city** _____

MORETTI S.P.A.

Via Bruxelles, 3 - Meleto 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

www.morettispa.com email: info@morettispa.com

 **MADE IN P.R.C.**

MORETTI S.P.A.

Via Bruxelles, 3 - Meleto
52022 Cavriglia (Arezzo)

Tel. +39 055 96 21 11
Fax. +39 055 96 21 200

www.morettispa.com
info@morettispa.com